

Impulse® TENS D5

Digital TENS / EMS Stimulator



Impulse® TENS D5

PACKAGE INCLUDES

- 1 Impulse® TENS D5
- 4 2"x 2" electrodes
- 2 Lead wires
- 2 AA alkaline batteries
- 1 Instruction manual
- 1 Carrying pouch

SPECIAL FEATURES

- 5 TENS modes
- 4 EMS modes
- Low battery status indicator
- No-load detection
- Adjustable settings & Belt clip
- Patient Compliance Meter
- Flip style design protects screen
- HCPCS E0730 or E0745



GENERAL SPECIFICATIONS

SIZE:	5.5" x 2.5" x 1"
WEIGHT:	3.35 oz
POWER SOURCE:	2 AA Batteries, Type LR6
CHANNELS:	Dual (2)
WAVEFORM:	Symmetrical rectangular biphasic
PULSE RATE:	1 - 150 Hz (Hertz or pps) adjustable
PULSE WIDTH:	50 - 250 μ s (TENS), 50 - 300 μ s (EMS)
INTENSITY:	0- 100 mA peak to peak
TIMER:	Continuous, 15, 30, 45, 60 minutes
TOLERANCES:	+/- 10%

Does pain management need to involve drugs? Absolutely not! Pain can affect your mood, social events, and joyful activities with family & friends. Researchers agree that the use of an electrotherapy device can provide sufficient relief from neuropathy, muscle, or joint pain so you can live and enjoy life!

The Impulse® TENS D5 is a portable dual channel transcutaneous electrical nerve stimulator (TENS) for pain relief and an electronic muscle stimulator (EMS) used for stimulating healthy muscles in order to improve and facilitate muscle performance.

TENS modes:	EMS modes:
<p><u>P1 - Constant</u>, PR 120* PW 50** / Continuous stimulation.</p> <p><u>P2 - Burst 1</u>, PR 28, PW 150** / 2 burst per sec / 7 pulses per burst.</p> <p><u>P3 - Burst 2</u>, PR 80, PW 150** / 1 burst per 2 sec / 80 pulses per burst.</p> <p><u>P4 - PW Mod</u>, PR 120*, PW 250-50-250 / PW modulates over 5 sec & repeats.</p> <p><u>P5 - PR Mod</u>, PR 100-20-100, PW 150** / PR modulates over 5 sec & repeats.</p> <p>* Adj. PR from 1-150 Hz</p> <p>** Adj. PW 50, 100, 150, 200, 250 μS</p>	<p><u>P6 - Cycled</u>, PR 35, PW 300** / 2 sec amp ON, 10 sec time ON, 2 sec ramp OFF, 30 sec rest-repeat.</p> <p><u>P7 - Reciprocal</u>, PR 40, PW 300** / 4 sec ramp ON, 10 sec time ON, 4 sec ramp OFF, 10 sec rest-repeat.</p> <p><u>P8 - Cycled</u>, PR 40, PW 300** / 4 sec ramp ON, 10 sec time ON, 4 sec ramp OFF, 10 sec rest-repeat.</p> <p><u>P9 - Cycled</u>, PR 20, PW 200 / 2 sec ramp ON, 5 sec time ON, 2 sec ramp OFF, 10 sec rest-repeat.</p> <p>** Adj. PW 50, 100, 150, 200, 250 μS</p>

The Impulse® TENS D5 incorporates a "Flip Style" design to protect the LCD screen and control buttons keeping the unit clean from dust and grim. The *Patient Compliance Meter* (PCM) feature can lock-in the most frequently used or favorite programs set by the physician while keeping track of the hours used.

This device comes with everything needed to begin and manage your therapy sessions.

Data recorded across a 500 OHM load resistance



1954 Kellogg Avenue
Carlsbad, CA 92008
Ph: (800) 726-8367 USA
Ph: (760) 579-0801 INTL
Fx: (760) 929-9953
www.bmls.com
information@bmls.com



Cage Code 3R8H6

